


FYR Collection Kit Return Mailing Instructions

Kits will come in a FedEx Express box. Within the FedEx box you will find smaller brown boxes topped with FYR Diagnostics stickers – each box is a collection kit.

Within each kit you should receive labeled tubes with collection media, equal number of swabs, biohazard bags for return shipping, a scrap piece of paper for return shipping. You will also receive documents.

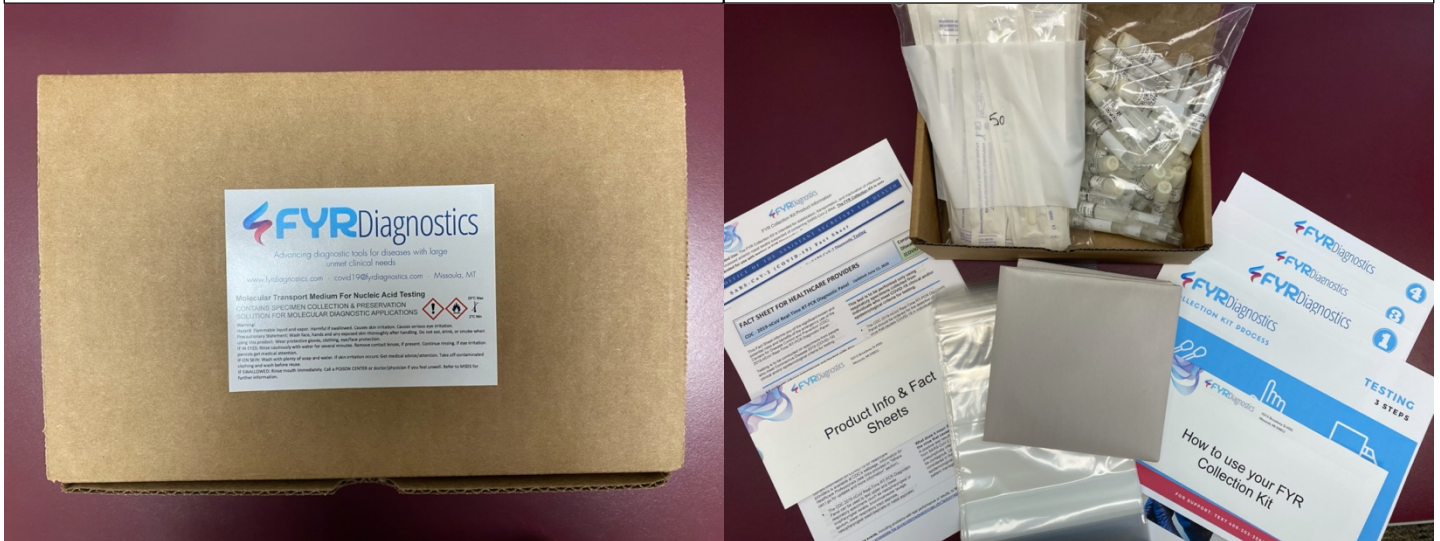

Empty FYR boxes to utilize for return shipping or utilize your own or a FedEx box. FedEx Express boxes are free to use as we ship Express. Order online or visit your nearest FedEx location to stock up.

After collecting samples, **package 5 tubes per provided biohazard bag** and always include the provided absorbent material. Make sure the tube caps are secured tightly!

Package the FYR box (or utilize a FedEx/your own box) so that the tubes won't rattle around to decrease the chance of them breaking. You can use the paper we included or supplement with your own.

Place the included return label on the top of the FYR box and make sure to seal the front of the box so that it doesn't open during shipping. Utilize your own tape and drop off at a FedEx location.



FYR includes return shipping labels in each order. If you require more return labels please text or call the FYR Help Line at (406) 303 3560 to discuss your needs and pricing. You are also welcome to create and use your own labels.